




S70 in-bra
breast pump

quick guide

quick guide

getting started

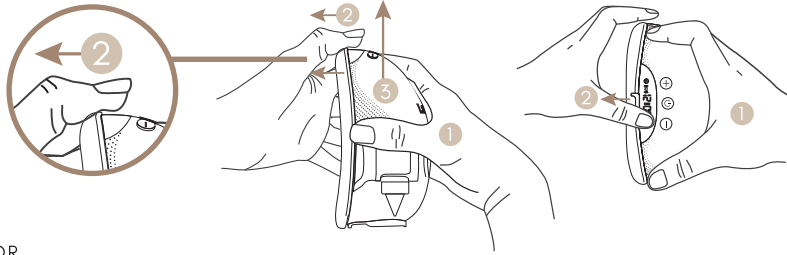

REMOVE PUMP MOTOR



- 1 PUT YOUR THUMB, INDEX & MIDDLE FINGER ON THE GRIP EITHER SIDE OF THE MOTOR
- 2 WITH YOUR OPPOSITE THUMB UNCLIP THE SPOUT OF THE MILK COLLECTOR FROM THE MOTOR
- 3 PULL THE MOTOR UPWARDS AND AWAY FROM MILK COLLECTOR

DISASSEMBLE

- 1 REMOVE DIAPHRAGM
- 2 REMOVE FLANGE BY RELEASING EDGE FROM MILK COLLECTOR. PUT YOUR FINGERS EITHER SIDE OF TUNNEL & PULL IN OPPOSITE DIRECTION
- 3 REMOVE VALVE FROM BOTTOM OF FLANGE TUNNEL

CHARGING


- 1 REMOVE USB C COVER
- 2 CONNECT USB C TO POWER SOURCE BATTERY ICON BARS WILL FLASH WHEN CHARGING. CHARGE UNTIL FULLY CHARGED & BATTERY ICON IS SOLID

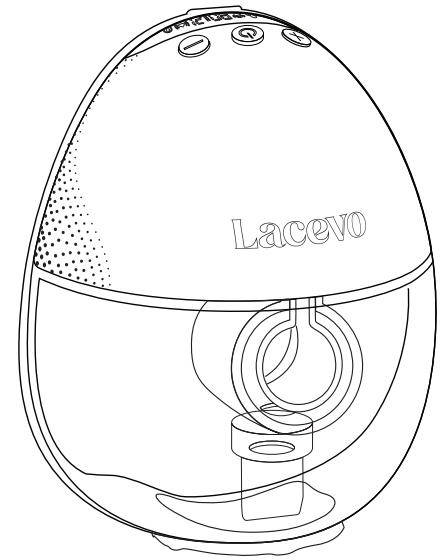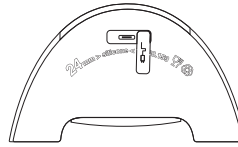

pump parts

DO NOT WASH

PUMP MOTOR

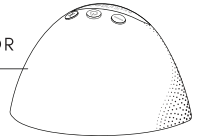

cleaning

FIRST USE

SOAK ALL WASHABLE PARTS (NOT MOTORS) IN BOILING WATER FOR AT LEAST 5 MINUTES. REMOVE AND AIR DRY

REGULAR CLEANING

USE CLEANING BRUSHES TO CLEAN WASHABLE PARTS WITH WARM WATER & MILD DISH SOAP. RINSE, REMOVE AND AIR DRY.

WASH WITH BOILING WATER, FROM TIME TO TIME FOR DEEP CLEAN

DO NOT PUT IN DISHWASHER OR SANITISER AS IT MAY SHORTEN THE LIFE OF YOUR PARTS. WE RECOMMEND YOU REPLACE SILICONE PARTS EVERY 1-3 MONTHS DEPENDING ON FREQUENCY OF USE




PUMP

WIPE WITH 70-90% ETHYL OR ISOPROPYL ALCOHOL


WASHABLE PARTS

DIAPHRAGM

24MM FLANGE

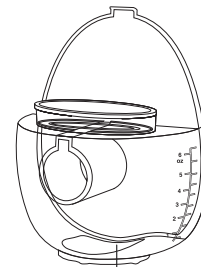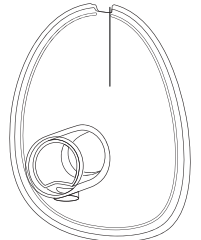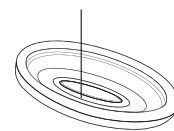


MILK COLLECTOR

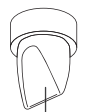


VALVE


assembly

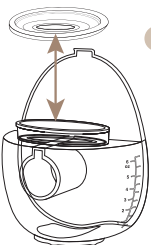

- 1 MAKE SURE YOUR HANDS ARE CLEAN & DIAPHRAGM IS DRY BEFORE ATTACHING IT TO THE TOP OF THE MILK COLLECTOR

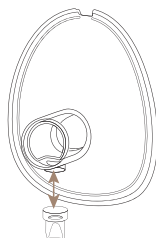


- 2 CONNECT VALVE TO THE OPENING ON THE BOTTOM OF THE FLANGE TUNNEL.

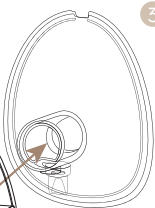


- 3 INSERT TUNNEL OF THE FLANGE INTO THE TUNNEL OF THE MILK COLLECTOR. PUSH FIRMLY AS FAR AS POSSIBLE, MAKING SURE THE SPOUT IS CENTRED

MAKING SURE THE SPOUT IS ALIGNED, WORK YOUR FINGERS AROUND THE FLANGE SO THAT IT SEALS PROPERLY TO THE MILK COLLECTOR

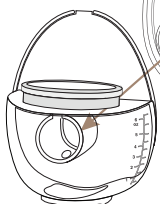

- 4 CONNECT PUMP MOTORS

WITH ONE HAND PUT YOUR THUMB, INDEX & MIDDLE FINGER ON THE GRIP EITHER SIDE OF THE MOTOR WITH YOUR OTHER HAND HOLDING THE BASE OF THE MILK COLLECTOR, PUSH THE MOTOR DOWN ONTO THE DIAPHRAGM UNTIL IT CLICKS INTO PLACE

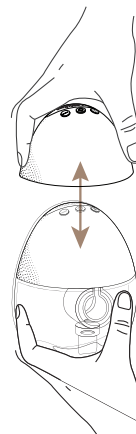

TIPS

IF PUMP MOTOR ISN'T GOING ON PROPERLY TAKE OFF AND CHECK THAT USB C COVER IS INSERTED FLAT AND THAT THE DIAPHRAGM IS CONNECTED EVENLY


operation

POWER ON/OFF

FOR ON PUMP OPERATION, LONG PRESS THE CENTRE BUTTON TO TURN THE PUMP ON/OFF. PUMP WILL START ON THE LAST USED MODE & LEVEL.

MODES

THERE ARE 4 MODES & 12 LEVELS. WE RECOMMEND STARTING ON STIMULATION MODE & THE LOWEST SETTING & INCREASE AS YOU FIND COMFORTABLE. TO ALTERNATE MODES SHORT PRESS THE CENTRE BUTTON

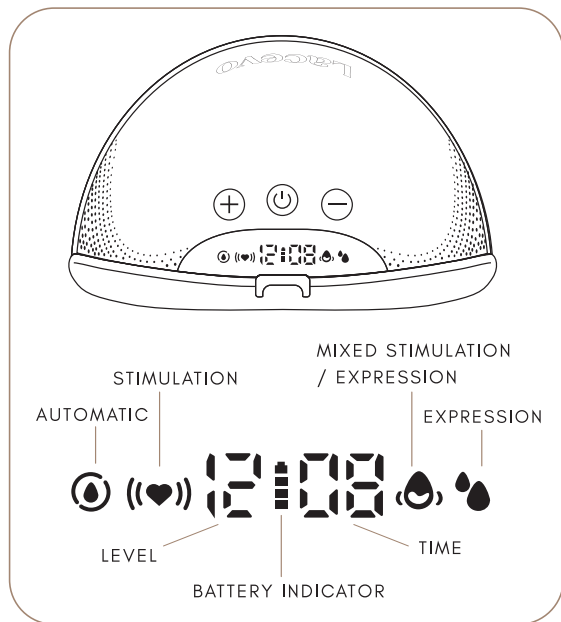


LEVELS

PRESS + TO INCREASE
PRESS - TO DECREASE

FILL SENSOR

MILK LEVEL SENSOR THAT WILL ALERT YOU WHEN YOUR MILK LEVEL IS GETTING HIGH. IF YOU ARE MOVING AROUND A LOT YOU CAN TURN IT OFF VIA THE APP TO AVOID IT MISREADING THE INCORRECT LEVEL

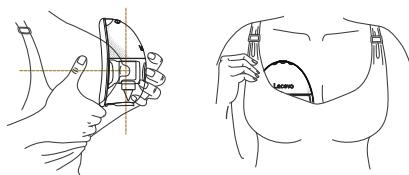


pumping

ALIGN YOUR NIPPLE TO THE CENTRE OF THE FLANGE OR SIZE INSERT TUNNEL. FIRMLY PRESS THE PUMP IN AN UPRIGHT POSITION AGAINST YOUR BREAST AND CAREFULLY PULL YOUR BRA OVER THE PUMP. WE RECOMMEND USING A FULL COVERAGE NURSING BRA TO ENSURE A FIRM FIT & THAT PUMP STAYS IN THE CORRECT POSITION

THE MOST IDEAL PUMPING POSITION IS SITTING TO ENSURE BREAST PUMPS ARE UPRIGHT. IF PUMPING ON THE GO PLEASE REMEMBER TO KEEP UPRIGHT SO MILK DOESN'T TIP OUT OF TOP SPOUT.

THE PUMP WILL AUTOMATICALLY SHUT OFF AFTER 30 MINUTES.

ONCE YOUR SESSION HAS FINISHED REMOVE YOUR PUMPS FROM YOUR BRA.

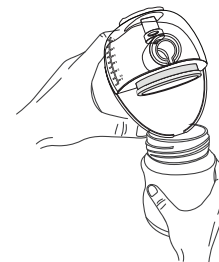

REMOVE THE PUMP MOTOR FROM THE MILK COLLECTOR AND CAREFULLY POUR THE MILK FROM THE TOP SPOUT INTO A CLEAN BOTTLE OR PREFERRED MILK STORAGE CONTAINER. STORE MILK SAFELY AS PER CDC GUIDELINES

NEVER POUR MILK OUT WITH MOTOR STILL CONNECTED

operation guide video

SCAN CODE TO REFER TO OPERATION GUIDE VIDEO MORE INFORMATION ON USING OUR S70 PUMP, APP & ACCESSORIES OR CHECK OUT LACEVO.COM

